



# MULTI-MAM SUMMARY OF STUDIES

## Contact details

BioClin BV

Eveline van Dijk

[e.vandijk@bioclin.com](mailto:e.vandijk@bioclin.com)

+ 31 (0)15 2512 895

JUNE 2014  
BIOCLIN BV

# 2004, Multi-Mam Compresses: evaluation of the relief and treatment of nipple pain in breastfeeding women

## Introduction

For a post marketing survey, Report Forms were provided to several Swiss and German midwives and lactation consultants for distribution to breastfeeding mothers using Multi-Mam compresses. In the Report Forms the users of Multi-Mam Compresses could fill in a number of questions, the results of the application of Multi-Mam Compresses and remarks on the product.


227 Report Forms were collected over a period of six months. The age distribution was 38 mothers of 35-42, 94 mothers of 30-35, 64 mothers of 25-30 and 31 mothers of 20-25. The complaints were noted as; very sensitive, sore nipples, inflamed, damaged nipples and bleeding fissures.

## Results:

The Result Forms asked for a 2 days result of the application of the compresses, therefore emphasizing the expectation of a quick result. With a drop of the average pain score of 7 to 3 this was indeed shown. It was shown that the quickest results were obtained when the nipple pain was immediately treated with the Multi-Mam Compresses; within 1-2 days after start of complaints.

### Evaluation of the relief and treatment of nipple pain in breastfeeding women:

Reduction of complaint score during treatment  
(number of participants = 227):

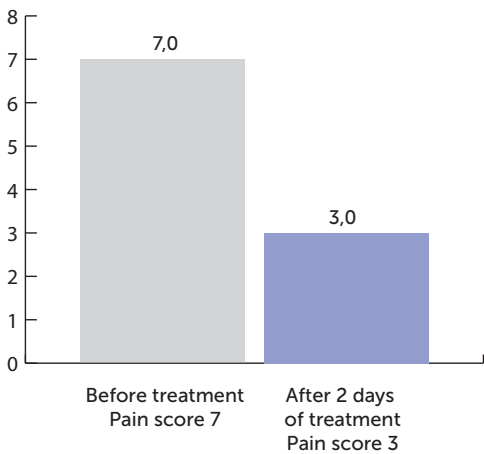

Pain score 0 - 10:  
0 = no pain and 10 = extreme pain

## Conclusion

The results of the evaluation of Multi-Mam Compresses for the relief and treatment of nipple pain with 227 mothers are quite impressive with respect to the complete resolve within 2 days.

A significant number of women were extremely satisfied with the relief of pain, inflammation and nipple damage, which greatly contributed to the continuation of breastfeeding.

# 2013, Multi-Mam BabyDent survey

## Introduction


In August 2013, a survey among 33 mothers with teething babies was performed. The mothers received a tube of Multi-Mam BabyDent for their teething baby. The average use was 2 times a day. Thereafter the mothers answered a questionnaire.

## The main results are:




- 81% of the mothers considered that Multi-Mam BabyDent relieved teething pain quickly.
- 87% of the mothers were very satisfied with Multi-Mam BabyDent as a product.
- 93% of the mothers welcomed and appreciated the fact that Multi-Mam BabyDent does not contain any anaesthetic ingredients.
- 81% of the mothers will recommend Multi-Mam BabyDent to others.

I would gladly recommend Multi-Mam BabyDent to other mothers

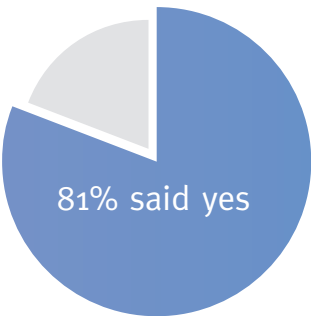

Do you agree that Multi-Mam BabyDent quickly relieved the teething pain of your child?

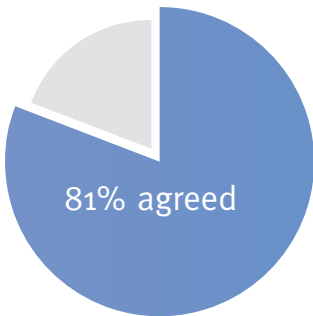

How do you rate Multi-Mam BabyDent on a rate from excellent to insufficient?

